



# RIVA CEM

RESIN MODIFIED GLASS IONOMER  
LUTING CEMENT



NO WASTAGE  
& THE BENEFIT  
OF A LIGHT  
TACK CURE

# EASY TO MIX PASTE/PASTE LUTING CEMENT FOR THE PERMANENT CEMENTATION OF INDIRECT RESTORATIONS

Riva cem is a self-curing, radiopaque, fluoride releasing paste/paste resin modified glass-ionomer luting material. It is indicated for permanent cementation of metal-based and strengthen-core ceramic restorations, and orthodontic appliances.



**EASY TO MIX  
PASTE/PASTE SYSTEM**



**GREATER STAIN RESISTANCE**

RIVA CEM IS AVAILABLE IN A UNIVERSAL LIGHT YELLOW SHADE IN DOUBLE BARREL SYRINGE WHICH CAN EASILY BE HAND MIXED.



# RIVA CEM

## 1 X 8.5G RIVA CEM SYRINGE



HIGH BOND STRENGTH



LIGHT TACK CURE  
OPTION FOR QUICK  
AND EASY REMOVAL  
OF EXCESS CEMENT



SUSTAINED FLUORIDE RELEASE

### KEY FEATURES

Light tack cure option for quick and easy removal of excess cement

High bond strength

Greater stain resistance

Sustained fluoride release

BPA free

Easy to mix paste/paste system

Light cure any excess material for 5 seconds per surface using a high power LED curing light (460-480nm emission). Then gently remove it using a scaler or explorer. Excess can also be easily removed during self-curing gel stage (when cement feels rubbery), after 1 min 45 seconds.


### SHEAR BOND STRENGTH [MPa]



# RIVA CEM

INDICATED MATERIAL	TYPE OF RESTORATION
PFM	Crowns & bridges
Resin	Crown, bridges, inlays, onlays & posts (fiber-reinforced)
Metals	Crown, bridges, inlays, & onlays, orthodontic appliances
Ceramics (high strength)	Crowns & bridges
All-ceramic	Inlays

## ORDER DETAILS

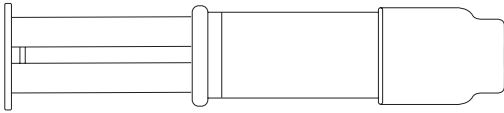

### SYRINGES

Riva Cem	8800700
1 x 8.5g riva cem syringe	

### TECHNICAL DATA

Working time	1'45"
Setting time	4'30"
Shear bond strength [MPa] - Dentin	13.03 ± 3.40
Shear bond strength [MPa] - Enamel	9.99 ± 3.17
Compressive strength [MPa]	108



YOUR SMILE.  
OUR VISION.

M100294 E

05/2023



+D036M1002941/

MADE IN AUSTRALIA  
by SDI Limited  
Bayswater, Victoria 3153  
Australia 1800 337 003  
[www.sdi.com.au](http://www.sdi.com.au)

**AUSTRIA** 00800 0225 5734  
**BRAZIL** 0800 770 1735  
**FRANCE** 00800 0225 5734  
**GERMANY** 0800 100 5759  
**ITALY** 00800 0225 5734

**NEW ZEALAND** 0800 808 855  
**SPAIN** 00800 0225 5734  
**UNITED KINGDOM** 00800 0225 5734  
**USA & CANADA** 1 800 228 5166