SDI

RIVA STAR AQUA

E  
N  
G  
L  
I  
S  
HINSTRUCTIONS FOR USE  
(CAPSULES)

Riva Star Aqua est un ammoniaque-free new generation desensitizer that immediately relieves tooth sensitivity.

## INDICATIONS FOR USE:

- Treatment of dental hypersensitivity.
- For use in adults over the age of 21.

## CONTRA-INDICATIONS

- Do not use in patients with ulcerative gingivitis, stomatitis or patients undergoing thyroid gland therapy.
- Do not use on patients with known sensitivity to silver or other heavy - metal ions, fluoride, potassium or iodine.

**Note:** If in doubt, consult a doctor or an allergy specialist before use. Patients with more than five affected teeth, patients having had full mouth gingivectomies and patients showing abnormal skin sensitization in daily circumstance are recommended for exclusion.


## PROCEDURE:

1. Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
2. Assess condition of teeth. Only teeth with sound dentine and enamel should be treated.
3. Isolate area to be treated using cotton rolls.
4. Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site
5. **Riva Star Aqua Step 1:** Using the **silver blue** brush provided, pierce through the foil of the **silver blue** capsule and carefully apply solution to treatment site only.
6. **Riva Star Step 2:** Immediately after, use **green** brush provided, pierce through the foil of the **green** capsule and apply a generous amount of the solution to treatment site **until the creamy white precipitate turns clear** (if this precipitate does not turn clear staining may result). **Note:** Apply same solution from **green** capsule to any other tooth surfaces that may have come into contact with the solution from Riva Star Aqua Step 1 (**silver blue** capsule).
7. Blot dry.
8. Remove all protective/isolation materials used in the mouth.
9. Discard used capsules and brushes in accordance with local regulations.

## DOSAGE:

- Each Riva Star Aqua Step 1 and Step 2 capsule can treat up to 5 teeth per patient.
- Limit of one (1) Riva Star Aqua Step 1 & one (1) Riva Star Step 2 capsule per visit. If needed, a second application of Riva Star Aqua may be administered after 1 week.

Fig.1 How to pierce capsule

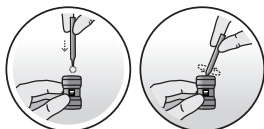1. Use colour coded brush to pierce foil on capsule.
2. In a circular motion, push foil to the edge of the opening.

## PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Due to potential staining, it is recommended that anterior teeth not be treated.
- Patient must be informed of potential staining prior to treatment.
- Avoid contact of Riva Star Aqua Step 1 solution (**silver blue** capsule) with oral soft tissue, eyes and skin. If accidental contact occurs, refer to "FIRST AID" below.
- Avoid contact of Riva Star Step 2 solution (**green** capsule) with eyes.
- Do not desensitize teeth with soft dentine as staining may occur.
- **Note:** If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self cure glass ionomer cement (such as Riva Self Cure, SDI Limited).
- Keep out of reach of children.
- Do not ingest.
- Riva Star Aqua Step 1 Solution (**silver blue** capsule) will stain (see "HANDLING STAINS").
- Wear gloves and eye protection when handling.
- Single use only.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Riva Star Aqua Step 1 Solution (**silver blue** capsule) contains 0.06g / mL fluoride ions.
- SDS is available at [www.sdi.com.au](http://www.sdi.com.au) or contact your SDI representative.

## STORAGE AND HANDLING:

- Do not pierce foil of Riva Star Aqua **silver blue** capsule or Riva Star **green** capsule until ready to use.
- Store in a cool and dark place at temperatures between 2° and 25°C (35° - 77°F).
- Use at room temperature between 20° and 25°C (68° - 77°F).

## FIRST AID:

- **Eyes (contact):** Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes.
- **Skin/oral soft tissue (contact):** Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- **Ingestion:** Not expected due to dose volume. If affected, do NOT induce vomiting. Give a glass of milk or water. For advice contact a Poisons Information Centre or a doctor.
- **Inhalation:** Not expected due to dose volume. If affected, remove from contaminated area.

## HANDLING STAINS:

- **Skin:** In case of skin contact with Riva Star Aqua Step 1 (**silver blue** capsule), apply Riva Star Step 2 (**green** capsule) with gentle rubbing. Stain will slowly disappear. Otherwise, if Riva Star Step 2 is not applied, the stain should disappear gradually after a few days.
- **Clothing:** Remove contaminated clothing and soak in Napisan\* (or other fabric stain remover) overnight and wash per normal.
- **Hard surfaces:** Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan\* is not the registered trademark of SDI Limited.

## PACKAGING:

- Each kit contains:
- 10 x Riva Star Aqua Step 1 (**Silver blue** capsule)
- 10 x Riva Star Step 2 (**Green** capsule)
- Accessories

US Patents granted.

Made in Australia by SDI Limited  
3-15 Brunston Street  
Bayswater, Victoria 3153  
Australia

Australia 1 800 337 003  
Austria 00800 0225 5734  
Brazil 0800 770 1735  
France 00800 0225 5734  
Germany 0800 1005759  
Italy 00800 0225 5734  
Spain 00800 0225 5734  
United Kingdom 00800 0225 5734  
USA & Canada 1 800 228 5166  
[www.sdi.com.au](http://www.sdi.com.au)  
[www.rivastar.com](http://www.rivastar.com)

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Napisan\* no es una marca registrada de SDI Limited.