SDI RIVA STAR AQUA

ENGLISH

INSTRUCTIONS FOR USE (BOTTLE)

Riva Star Aqua is an ammonia free, water based dental cariostatic and desensitizing agent that detects and arrests carious lesions.

INDICATIONS FOR USE:

- A. Desensitising cervical tooth hypersensitivity.
- B. Arresting Caries
- C. Detection of carious lesions

CONTRA-INDICATIONS

Do not use on patients undergoing thyroid gland therapy, who have a history of chemical allergies such as silver, fluoride, potassium and iodine. Note: If in doubt, consult a doctor or an allergy specialist before use.

Note: If the product is used in carious lesions or demineralized areas, the treatment site may present a darkening effect. The patients should be informed before treatment.

PROCEDURE:

A. DENSENITISING CERVICAL TOOTH HYPERSENSITIVITY:

- Assess the condition of the teeth. Only treat teeth with sound dentin and enamel unless a restorative material is used to cover the treated area.
 - Using a non-fluoride prophylaxis paste in a rubber cup, clean the tooth/teeth to be treated and abutting teeth.
 - Isolate area to be treated with cotton rolls.
 - Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site.
 - Riva Star Aqua Step 1:** Dispense one (1) drop of solution (blue label) onto non-absorbent mixing pad or dappen dish.
 - Carefully apply solution to treatment site only, a medium sized micro brush, such as Points (SDI Limited) is recommended.
 - Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (green label) onto fresh non-absorbent mixing pad or dappen dish.
 - Apply a generous amount of the Riva Star Step 2 (green label) solution to treatment site, a medium sized micro brush, such as Points (SDI Limited) is recommended. Apply same solution to any other tooth surfaces that may have come into contact with the Riva Star Aqua Step 1 solution (blue label). **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 solution (green label) until it turns clear.
 - Blot dry.
 - Remove all protective/isolation materials used in the mouth.
- IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (Eg: tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.
- Clean dappen dishes and discard used mixing pads and micro-brushes in accordance with local regulations.

B a). ARRESTING CARIES WITHOUT RESTORING TOOTH:

Note: If used on carious lesions or demineralised dentine, treatment site will stain or slightly discolour. Patients must be informed of this prior to treatment. Ideally keep treatment away from aesthetic zone.

- Using a non-fluoridated prophylaxis paste in a rubber, clean the teeth to be treated and the adjacent teeth.
 - Isolate area to be treated with cotton rolls.
 - Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site.
 - Remove any food debris using an excavator.
- NOTE 1:** Make sure that the region where the Riva Star Aqua will be applied is well insulated to prevent product flow to the adjacent teeth.
- NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.
- Riva Star Aqua Step 1:** Dispense one (1) drop of solution (blue label) onto non-absorbent mixing pad or dappen dish.
 - Carefully apply solution to treatment site only keeping in contact with the caries lesion for 60 seconds, a medium sized micro brush is recommended, such as Points (SDI Limited).
 - Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (green label) onto fresh absorbent mixing pad or dappen dish.
 - Apply a generous amount of the solution to treatment site, a medium sized micro brush is recommended, such as Points (SDI Limited). Apply same solution to any other tooth surfaces that may have come into contact with the Riva Star Aqua Step 1 solution (blue label). **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 solution (green label) until it turns clear. (It is possible that a residual yellowish matt surface may remain).
 - Remove all protective/isolation materials used in the mouth.
 - Blot dry.

IMPORTANT : Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

- Reschedule patient to return for permanent restoration procedure. For deciduous teeth, wait for tooth re-eruption.
- Clean dappen dishes and discard used mix pads and micro-brushes in accordance with local regulations.

B b). ARRESTING CARIES AND RESTORING TOOTH:

Note: If the product is used in carious lesions or demineralized areas, at the treatment site have a darkening effect. The patients should be informed before treatment. Preferably, keep the treatment away from the aesthetic area.

- Using a non-fluoridated prophylaxis paste in a rubber, clean the teeth to be treated and the adjacent teeth.
 - Isolate area to be treated with cotton rolls.
 - Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site.
- NOTE 1:** make sure that the region where the Riva Star Aqua will be applied is well insulated to prevent product flow to the adjacent teeth.
- NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.
- To help prevent staining at the restoration margin, its recommended to create a moat (trough) around the dentine enamel junction with a #3 round bur into sound dentine.
 - Apply 37% phosphoric acid, e.g. Super Etch (SDI Limited), to site for 5 seconds - wash with water then dry.
 - Before using the brush, tap the bottom of the bottle on a hard surface to minimize the liquid coming out of the dropper tip.
 - Riva Star Aqua Step 1:** Dispense one (1) drop of solution (blue label) onto non-absorbent mixing pad or dappen dish.
 - Carefully apply solution to treatment site only, a medium sized micro brush is recommended, such as Points (SDI Limited).
 - Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (green label) onto fresh non-absorbent mixing pad or dappen dish.
 - Apply a generous amount of the solution to treatment site, a medium sized micro brush is recommended, such as Points (SDI Limited). Apply same solution to any other tooth surfaces that may have come into contact with the Riva Star Aqua Step 1 solution (blue label). **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 solution (green label) until it turns clear. (It is possible that a residual yellowish matt surface may remain).
 - Wash thoroughly with water for at least 10 seconds, and air dry. **Note:** Do not desiccate surface if a glass ionomer is to be used.
 - Discard used mixing pads and brushes in accordance with local regulations.
 - Clean dappen dishes and discard used mixing pads and micro brushes in accordance with local regulations.

a) Place self-cure glass ionomer cement such as Riva Self Cure (SDI limited).

Note: Polylytic conditioning is not required before placing glass ionomer.

OR

- If a composite material, such as Aura Bond Fill (SDI Limited) is to be used to restore the tooth,** first a conventional glass ionomer cement should be placed as a liner, followed by a glass ionomer based bonding agent such as Riva Bond LC (SDI Limited) or resin-based adhesive, such as Zipbond Universal (SDI limited). As a final layer, a resin-based composite, such as Aura Bulk Fill (SDI Limited) may be placed to finish off the restoration.

Note: If a resin based composite is directly placed over the Riva Star site, staining may occur due to any free silver ions becoming light activated, hence darkening.

IMPORTANT: Ensure that both Step 1 and Step 2 bottle surfaces (Eg: tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

- Remove all protective/isolation materials used in the mouth.

C. DETECTION OF CARIOUS LESIONS

Note: If the product is used in carious lesions or demineralized areas, the treatment site it may present a darkening effect. The patient should be informed before treatment:

- Isolate area to be treated with cotton rolls.
 - Protect any exposed gingival tissues and lips with petroleum jelly/ cocoa butter taking care not to contaminate treatment site.
- NOTE 1:** Make sure that the area where the Riva Star Aqua will be applied is well insulated to prevent product flowing to the adjacent teeth.
- NOTE 2:** If the product seeps into a tooth that has decayed tissue, it may stain.
- Riva Star Aqua Step 1:** Dispense one (1) drop of solution (blue label) onto non-absorbent mixing pad or dappen dish.
 - Carefully apply solution to site to be diagnosed and leave for 1 minute, a medium sized micro brush is recommended, such as Points (SDI Limited).
 - Riva Star Step 2:** Immediately after, dispense two (2) drops of solution (green label) onto fresh non-absorbent mixing pad or dappen dish.
 - Apply a generous amount of the solution to treatment site, a medium sized micro brush is recommended, such as Points (SDI Limited). Apply same solution to any other tooth surfaces that may have come into contact with the Riva Star Aqua Step 1 solution (blue label). **Note:** Treatment surface initially appears creamy white, keep applying Riva Star Step 2 solution (green label) until it turns clear.
 - Wash thoroughly with water for at least 10 seconds and air dry. **Note:** Do not desiccate surface if a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited), is to be used.
 - Remove all protective/isolation materials used in the mouth.
 - Discard used mixing pads and brushes in accordance with local regulations.
- IMPORTANT:** Ensure that both Step 1 and Step 2 bottle surfaces (Eg: tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.
- Clean dappen dishes and discard used mix pads and brushes in accordance with local regulations

Note: Treated sites should begin to turn brown within 24 hours where there are early stages of carious lesions. A dental curing light can be used to accelerate the visible colour change. Stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited).

DOSAGE:

- One (1) drop of Riva Star Aqua Step 1 and two (2) drops of Riva Star Step 2 can treat up to 5 teeth per patient.
- Limit of one (1) drop of Riva Star Aqua Step 1 and two (2) drops of Riva Star Step 2 per visit. If needed a second application of Riva Star Aqua may be administered after 1 week.

PRECAUTIONS:

- For professional use only.
- Do not repeat treatment on pregnant or lactating women.
- Due to potential staining, it is recommended that anterior teeth not be treated unless diagnosing for caries. When diagnosed with caries, please follow procedure B (Arresting Caries) of this Instructions for Use.
- Patients must be informed of potential staining prior to treatment
- Avoid contact of solution from Riva Star Aqua Step 1 bottle (blue label) with oral soft tissue, eyes, skin and clothing. If accidental contact occurs refer to the "FIRST AID" below.
- Avoid contact of solution from Riva Star Step 2 bottle (green label) with eyes.
- When used as a desensitiser, do not desensitise teeth with carious lesions as staining may occur. **Note:** If it does occur, stained sites can be reduced by using a judicious polishing procedure with phosphoric acid impregnated pumice powder and then covered with a self-cure glass ionomer cement, such as Riva Self-Cure (SDI Limited).
- Keep out of reach of children.
- Do not ingest.
- Solution from Riva Star Aqua Step 1 bottle (blue label) will stain (see "HANDLING STAINS").
- Wear gloves and eye protection when handling.
- Do not use after expiry date.
- Caution: Federal Law restricts this device to sale by or on the order of a dentist.
- Solution from Riva Star Aqua Step 1 bottle (blue label) contains 0.06g / mL fluoride ions.
- SDI is available at www.sdi.com.au or contact your SDI representative.

STORAGE AND HANDLING:

- Store bottles upright in the packaging provided.
- Store and use at room temperature between 20° and 25°C (68° - 77°)
- Do not refrigerate products
- Replace cap on bottle and seal tightly immediately after use.

IMPORTANT : Ensure that both Step 1 and Step 2 bottle surfaces (Eg : tip, neck, thread and inside the cap) are wiped thoroughly with an absorbent paper towel to remove any excess solution and prevent build up of dried material.

FIRST AID:

- Eyes (contact): Hold eyelids apart and flush the eye continuously with running water. Continue flushing until advised to stop by a Poisons Information Centre or a doctor, or for at least 15 minutes. Seek medical attention if symptoms persist.
- Skin/oral soft tissue (contact): Remove contaminated clothing, flush area with running water (in case an eschar is formed, this should disappear within 24-48 hours).
- Ingestion: Do NOT induce vomiting. Give a glass of milk or water. Seek immediate medical attention.
- Inhalation: If affected, remove from contaminated area.

HANDLING STAINS:

- Skin: In case of skin contact with Riva Star Aqua Step 1 solution (blue label), apply Riva Star Step 2 solution (green label) with gentle rubbing. Stain will slowly disappear. Otherwise, if Riva Star Step 2 solution is not applied, the stain should gradually disappear after a few days.
- Clothing: Remove contaminated clothing and soak in Napisan* (or other fabric stain remover) overnight and wash per normal.
- Hard surfaces: Immediately apply household bleach (sodium hypochlorite) then wipe off with water. For stubborn stains, saturate a paper napkin with household bleach and place over affected area. Leave overnight and mop up with water.

Napisan* is not the registered trademark of SDI Limited.

PACKAGING:

- Each Kit contains:
 - 1 x Riva Star Aqua Step 1 (blue label) bottle 1.5mL
 - 1 x Riva Star Step 2 (green label) bottle 3.0mL

US Patents granted.

Made in Australia by SDI Limited
3-15 Brunsdon Street
Bayswater, Victoria 3153, Australia

Australia 1 800 337 003
Austria 00800 0225 5734
Brazil 0800 770 1735
France 00800 0225 5734
Germany 0800 1005759
Italy 00800 0225 5734
Spain 00800 0225 5734
United Kingdom 00800 0225 5734
USA & Canada 1 800 228 5166
www.sdi.com.au
www.rivastar.com

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Napisan* no es una marca registrada de SDI Limited.