



HIGHLIGHTS

January | February 2025



TOOTH WHITENING

7+3^{*}

BUY 7, GET 3 FREE

TAKE-HOME WHITENING

POLA DAY, POLA NIGHT, POLA DAY CP, POLA LUMINATE, POLA FOR ALIGNERS AND POLA LIGHT



POLA DAY

HYDROGEN PEROXIDE BASED ADVANCED TRAY TOOTH WHITENING SYSTEM

- Available in **6% 7.5%** and **9.5% hydrogen peroxide** applications
- From 30 minutes once a day

4 X 1.3g MINI KIT	MRP \$41.47
10 X 1.3g SYRINGE KIT	MRP \$101.57
50 X 3g DISPENSER PACK	MRP \$564.30



POLA NIGHT

CARBAMIDE PEROXIDE BASED ADVANCED TRAY TOOTH WHITENING SYSTEM

- Available in **10% 16% 18%** and **22% carbamide peroxide**
- From 45 minutes once a day

4 X 1.3g MINI KIT	MRP \$41.47
10 X 1.3g SYRINGE KIT	MRP \$101.57
50 X 3g DISPENSER PACK	MRP \$564.30



POLA DAY CP

SUPER FAST TRAY TOOTH WHITENING SYSTEM!

- Available in **35% carbamide peroxide**
- From just 15 minutes per day

4 X 1.3g MINI KIT	MRP \$41.47
10 X 1.3g SYRINGE KIT	MRP \$101.57
50 X 3g DISPENSER PACK	MRP \$564.30



POLA LUMINATE

BRUSH ON & GO!

- Available in **6% hydrogen peroxide**, low viscosity fast drying gel
- No trays required - quick and easy professional brush on tooth whitener
- Apply twice a day for just 30 minutes

DENTIST PRICE \$27.30
RECOMMENDED PATIENT PRICE \$75.00



POLA LIGHT

COMPLETE YOUR WHITENING OFFERING WITH POLA LIGHT

- Available in **6% and 9.5% hydrogen peroxide**
- No custom trays required
- Reduced patient consult time
- Entry level whitening option
- Great front desk conversation starter

DENTIST PRICE \$109.15
RECOMMENDED PATIENT PRICE \$189.00




POLA FOR ALIGNERS

STRAIGHTEN AND BRIGHTEN TOGETHER

- Available in **10% carbamide peroxide**
- The greatest patient comfort in a whitening kit
- Contains potassium nitrate and fluoride to reduce sensitivity for long term use
- High viscosity gel ensures it can be easily and securely placed into the aligner tray, staying in place

POLA FOR ALIGNERS MRP \$87.10

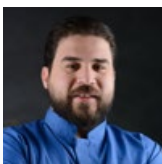

Dr. Mohamed Abdulmonem Abdulsalam, Egypt

Pola Light 6%



TOOTH WHITENING

POLA RAPID SUPER. FAST. WHITENING.



POLA RAPID

Pola Rapid is the new in office whitening treatment that safely whitens teeth in a 24 minute application time.

SUPERIOR HANDLING AND FAST REMOVAL

Pola Rapid's higher viscosity gel enables faster application and removal due to its specially formulated non-stick attributes. Pola Rapid can be extruded quickly and easily and remains on teeth when applied.

ENHANCED BLUE GEL FOR FASTER APPLICATION

Pola Rapid uses an advanced blue gel for ease of visibility and application. The easy to apply non-stick gel enables faster application and removal.

MINIMAL CHAIR TIME WITH IMMEDIATE RESULTS

A unique **38% hydrogen peroxide** formulation that releases active ingredients immediately. In only one visit, Pola Rapid initiates a powerful whitening treatment designed to whiten teeth in 24 minutes. No pre-desensitiser step is required.

POLA RAPID 3 PATIENT KIT	MRP \$464.49
POLA RAPID 3 PATIENT KIT WITH OPTRAGATE	MRP \$494.74
POLA RAPID 1 PATIENT KIT	MRP \$164.64
POLA RAPID 1 PATIENT KIT WITH OPTRAGATE	MRP \$176.14



Image source: Dr Miles Cone, USA.

PURCHASE 2 POLA RAPID
3 PATIENT KITS GET 3
POLA LIGHT 9.5% KITS
FREE*

TOTAL VALUE
\$327.45*



SDI | STELA

THE FUTURE OF COMPOSITES



- ✓ Unlimited depth of cure (self cure)
- ✓ Gap-free interface
- ✓ Only 2 steps: 15 sec prep
- ✓ High strength
- ✓ Choose: capsule or automix



Cavity prepared and ready to be isolated with rubber dam



Final aspect after finishing and polishing

Images courtesy of Prof Dr Rocio Lazo - Peru

STELA AUTOMIX 1x 8g SYR REFILL	MRP \$191.26
STELA AUTOMIX INTRO KIT	MRP \$231.66
STELA CAPSULE REFILL 50 CAPS	MRP \$278.08
STELA CAPSULES INTRO KIT	MRP \$138.61

STELA, LUNA 2 & LUNA FLOW

BUY 4 - GET 1 FREE



LUNA 2

Luna 2 is the next generation Luna universal composite. Luna 2 has an improved unique blend of fillers that were researched and developed at SDI to deliver enhanced handling, very high radiopacity and reliable mechanical properties. Luna 2 is BPA free for health-conscious patients. Luna 2 makes placing a composite easier than ever.


LUNA 2 4g SYRINGE	MRP \$111.56
LUNA 2 20 X 0.25g COMPLETS	MRP \$144.86



Scan QR code for more information



 Images courtesy of Dr Neus Sancho Lecina - Spain

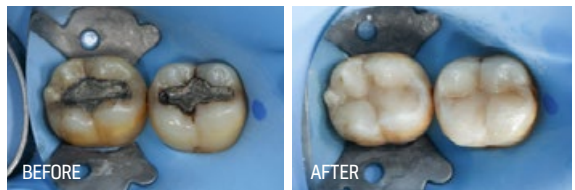

 Images courtesy of Dr Michael Chan - Australia



LUNA FLOW

Luna Flow is an extra strong flowable composite available in two viscosities that offer excellent mechanical and optical properties.

With a blend of fillers in different ultrafine sizes to optimize handling, mechanical and optical properties, Luna Flow offers high radiopacity, control over the application, easy and accurate shade matching with colour stability, excellent strength and wear resistance.

LUNA FLOW 2g SYRINGE	MRP \$58.43
----------------------	-------------



Scan QR code for more information

RESTORATIONS WITH LUNA FLOW COMBINED WITH LUNA 2



 Images courtesy of Dr Fay Goldstep & George Freedman - Canada



 Images courtesy of Dr Jose Amengual Lorenzo - Spain

GLASS IONOMERS



RIVA SELF CURE

BULK FILL GLASS HYBRID RESTORATIVE SYSTEM

- Contains SDI's proprietary **ionglass™**, a high ion releasing, radiopaque, bioactive glass that remineralises and prevents secondary caries formation
- Easy Activation Capsule - 70% easier to activate
- BPA & HEMA free
- Very high fluoride releasing and recharging ability to create more acid resistant restored teeth
- Ideal dentin replacement material
- Can be bulk filled

RIVA SELF CURE BOX OF 50 CAPS MRP \$270.15

BUY A BOX OF RIVA SELF CURE
GET A RIVA STAR AQUA
BOTTLE KIT FOR ONLY

\$90.00

SAVE \$54.36



NEXT GENERATION SDF:
RIVA STAR AQUA



SDI continues to innovate SDF solutions by releasing Riva Star Aqua, a next generation water based silver fluoride solution that is also effective in arresting carious lesions as an SDF solution, with additional patient benefits.

RIVA STAR AQUA BOTTLE KIT MRP \$158.79



Geoff Knight

SMART Technique

Current Concepts of Clinical Care

Silver modified ART (SMART) is becoming the gold standard of caries management. Riva Star/Riva Star Aqua is an adjunct to SDF/SF treatment that enables dentists to heal caries in deciduous and permanent teeth in a single visit without pain, tissue removal or staining.

RIVA STAR HAS THE FOLLOWING ADVANTAGES

- 1. FASTER** Clinicians have to wait while SDF/SF seeps into caries while Riva Star/Riva Star Aqua creates an immediate chemical reaction on the tooth surface that releases large amounts of fluoride and silver iodide.
- 2. "LESS TOOTH REMOVAL"** * As fluoride from Riva Star/Riva Star Aqua penetrates further into caries than SDF/SF only the removal of a minimal amount of broken down dentin is required.
- 3. "DOUBLE THE FLUORIDE RELEASE"** * When SDF/SF reacts with KI, fluoride ions are released into carious dentin at twice the concentration and penetrate much further into the caries than fluoride ions bound to silver in SDF.
- 4. "FORMS A BACTERICIDAL SHIELD"** * The reaction of SDF/SF and KI forms a bactericidal shield of AgI over the entire surface of carious dentin to a depth of 50 microns.
- 5. NO STAINING** As long as all the free silver ions have reacted, silver iodide is a creamy white precipitate that stabilizes as a tooth colored precipitate once it becomes bound to protein within the carious dentin and enamel.

Clinical Technique

1. Gain access to caries by removing an existing failed restoration as required.
2. Prepare the surface by disrupting any biofilm or badly broken down caries with a dental explorer.
3. It is important to etch the carious surfaces for 15 seconds to clean out any biofilm in the dentinal tubules and expose carious collagen to enable the AgI to stabilize by bonding to the broken down protein. Wash and dry the preparation.
4. Apply SDF/SF to carious surfaces avoiding any adjacent soft tissues.
5. Immediately apply KI until the white precipitate becomes clear signalling the reaction of all free fluoride ions. This will drop the pH of the SDF/SF from 13 to 9.
6. Wash away the reactant and dry.
7. Apply Riva Self Cure or Riva Light Cure either as a restoration or a base for a composite resin overlay.

Although dentists have been aware of the benefits of SDF/SF for almost 50 years it has never found traction within the dental profession due to the staining associated with it; apart from limited treatment of deciduous teeth, root caries and some public health systems.



Initial Cavity



Immediately post placement




Immediately after restoration



4 months Xray post placement



5 months post placement

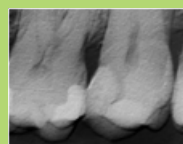

7 months post placement



14 months post placement

EQUIPMENT

RADII XPERT

SMARTER DESIGN.
OPTIMAL CURE EVERY TIME.

Radii Xpert is the ideal choice for restorations of all sizes, including bulk fill restorations that require a deep cure. The collimated beam ensures light intensity is maintained on the restoration being cured. Achieve a predictable cure, every time with Radii Xpert.



BUY FOR \$1800
get a bonus diagnostic tip

The Radii Xpert Diagnostic LED attachment provides transillumination to assist in early detection and diagnosis of clinical problems.

RADII XPERT
MRP \$2248.90



SAVE
\$840.67*

SEALANTS

CONSEAL F

LIGHT CURED FLUORIDE RELEASING RESIN PIT AND FISSURE SEALANT

- Extremely low viscosity - quickly and deeply penetrate small pits and tight fissures
- High Fluoride releasing
- BPA & HEMA free
- Accurate and controlled delivery with an ultrafine tip
- Opaque white shade
- 5mL bottles and 1g syringes

CONSEAL F SYRINGE KIT	MRP \$120.55
CONSEAL F BULK SYRINGE KIT	MRP \$344.34
CONSEAL F BOTTLE KIT	MRP \$197.45
CONSEAL F 5ML BOTTLE	MRP \$112.64

BUY 2 - GET 1 FREE*



BEFORE TREATMENT



AFTER TREATMENT



SDI

YOUR
SMILE.
OUR
VISION.

Call 1800 337 003
info@sdi.com.au
www.sdi.com.au
www.polawhite.com.au



New South Wales
Hany Gobran 0418 210 341
Elli Mouhtouris 0400 849 017
Allison Ohtback 0400 305 512

Western Australia
Toni Doherty 0448 086 925

South Australia/Northern Territory
Andriana Spithas 0417 080 771

Victoria
Jo Coyle 0419 313 765
Snezana Josifoski 0418 567 758
Shelby MacDonald 0408 174 626

Tasmania
Snezana Josifoski 0418 567 758

Queensland
Natalie Johnson 0409 807 290
Dora Zanin 0428 910 805

*Offers are valid 1st January - 28th February 2025. Offers cannot be combined. All prices include GST. Free goods will be of equal or lesser value. The above-mentioned offers are not applicable on corporate, government accounts or those customers who have received discounted pricing. Offers are valid in Australia only.